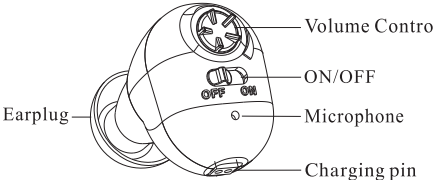

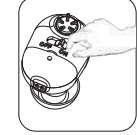

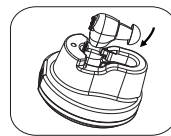
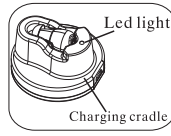
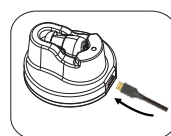




<p>Intended Use</p> <p>Thank you for the purchase of our hearing amplifier. If you find it's difficult to hearing very well from others, a hearing amplifier can help you regain your hearing so you can live the life you're used to.</p> <p>Wearing a hearing amplifier means rediscovering the pleasure of gathering with friends and family, feeling more confident at work, and enjoying movies, music and TV programs.</p> <p>Specification</p> <ol style="list-style-type: none"> 1.Max sound output:125±4dB 2.Sound gain: ≥30dB 3.Total harmonic wave distortion: ≤10% 4.Frequency range:450Hz~4000Hz 5.Input noise: ≤30dB 6.Voltage:D.C.3.7V 7.Current: ≤2mA 	<p>Instruction Diagram</p>  <p>"ON" is power on. "OFF" is power off. "↻" is volume grade.</p>	<p>How to use the hearing amplifier</p> <ol style="list-style-type: none"> 1.Insert the earplug into the hearing amplifier (Figure1). 2.Turn on the hearing amplifier. (Figure 2). 3.Put the hearing aid in your ear tightly and move it up and down to insure it on the right position (Figure 3). 4.If you feel the whistling sound, please make sure if you put the earplug tightly, or if you use an appropriate size earplug.    <p>Figure 1 Figure 2 Figure 3</p>	<p>How to maintain the hearing amplifier</p> <ol style="list-style-type: none"> 1. Please turn off the hearing amplifier when stop using. 2. Clean off dirt of the hearing amplifier case using a soft dry cloth. <p>The hearing aid doesn't work:</p> <ol style="list-style-type: none"> 1.Power is off, turn on the hearing amplifier. 2.Hearing amplifier is run out of the battery, please charge the hearing amplifier. <p>Sound is very light:</p> <ol style="list-style-type: none"> 1.Volume is too low, please increase the volume. 2.Battery is low, please charge the hearing amplifier.
--	---	---	--

<p>How to charge the hearing amplifier</p> <ol style="list-style-type: none"> 1.Turn off the hearing amplifier when you charge. 2.Put the hearing amplifier into the charging cradle(Figure 4、5),and insert the other end of the USB into the charging cradle(Figure 6). 3.Insert the charger plug into power supplier Socket(Figure 7),and start charge the hearing amplifier,and the LED light will be Red. Please recheck and connect it again if the LED light is Green. 4.The LED light on the charging cradle will turn red when charge,the LED light will turn green when full of charge. 5.The new hearing amplifier should be charged for 6 hours for first few times you use,after it only need about 2~3 hours for charge. 6.You must use appointed charger to charge the cell,and make sure that the power supply is the same as the rated power of charger. 	<p>How to charge the hearing amplifier</p> <p>7.The hearing amplifier is charged with specified charger (5V/50~500mA),the specified voltage is AC/DC 100~240V, unspecified power or voltage can cause a fire,electric shock,or hearing amplifier break down.</p>     <p>Figure 4 Figure 5</p> <p>Figure 6 Figure 7</p>	<p>Important Note</p> <ol style="list-style-type: none"> 1.Please don't use the hearing amplifier at a high volume for a long time. It will damage your hearing. 2.Please maintain the product all the time in order to avoid the earplug and microphone opening being blocked by foreign objects. 3.Please don't use when showering, swimming, raining or wet environment, please don't expose the hearing amplifier under sunshine, get close to a stove or other heat sources. 4.Please protect the hearing amplifier carefully and avoid dropping it on the floor. 5.Please use the specified battery and charger exclusively developed for the hearing amplifier. 6.Do not disassemble or modify the hearing amplifier. 7.Keep the hearing aid away from the children. 	<p>Operation Manual</p> 
---	---	---	--