

Intended Use


Thank you for the purchase of our hearing amplifier. If you find it's difficult to hearing very well from others, a hearing amplifier can help you regain your hearing so you can live the life you're used to.

Wearing a hearing amplifier means rediscovering the pleasure of gathering with friends and family, feeling more confident at work, and enjoying movies, music and TV programs.


Specification

1. Max sound output: 110 ± 5 dB
2. Max sound gain: $30 \text{ dB} \pm 5$ dB
3. Frequency range: 300~4500 Hz
4. Total harmonic wave distortion: $\leq 8\%$
5. Input noise: ≤ 32 dB
6. Current: ≤ 2 mA

Instruction Diagram



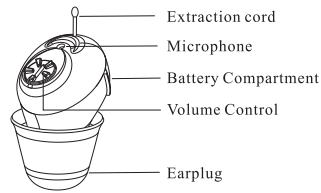


Battery: A10 Zinc Air Battery

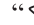- “+” Positive side of Battery
- “-” Negative side of Battery
- “” is Volume grade.


Operation

1. Insert the earplug to the hearing amplifier, put the hearing amplifier in your ear and move it on the right position, adjust the volume to get clear sound. (Figure A)
2. Open the battery compartment, put into the corresponding battery (A10 Zinc Air Battery) and close the battery compartment tightly. (Note: Open the battery compartment, it will stop working, close the battery compartment completely, it will start to work.) (Figure B)
3. Please take out the battery or turn off hearing amplifier when you stop to use the hearing amplifier. (Figure C)

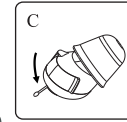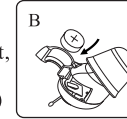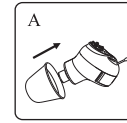

Operation

4. If you feel the whistling sound, please make sure if you put the earplug tightly, or if you use an appropriate size earplug. (Note: Be sure the hearing aid is in “Off” position or volume is in the minimum volume of “1”, otherwise the instant sound will make you feel uncomfortable.)
5. To adjust the sound more clearly, please rotate the volume control by the litter screwdriver. (Figure D)

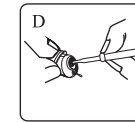

Maintenance

The hearing amplifier doesn't work.

Check as follows:

1. Hearing aid is power off.
2. The battery is in correct direction.
3. Run out of battery.
4. The earphone is damaged.
5. The earphone is broken.

Sound is very light

1. Volume is too low, please increase the volume.
2. Battery is low, please replace a new battery.

Clean off dirt of the hearing amplifier case using a soft dry cloth.


Note

1. Please don't use the hearing amplifier at a high volume for a long time. It will damage your hearing.
2. Please maintain the product all the time in order to avoid the earplug and microphone opening being blocked by foreign objects.
3. Please don't use when showering, swimming, raining or wet environment, please don't expose the hearing amplifier under sunshine, get close to a stove or other heat sources.
4. Please protect the hearing amplifier carefully and avoid dropping it on the floor.

Note

5. Please don't remove old batteries in the hearing amplifier for a long time in order to avoid possible erosion caused by battery leakage.
6. Please handle the old batteries environmentally friendly.
7. Keep the hearing amplifier away from the children.

Operation Manual

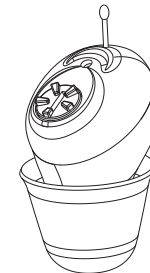

Rechargeable Body Aids

□ Instruction diagram



- ˘OFF˘: Power is off.
- ˘ON˘: Power is on.
- ˘N˘: Hearing aid frequency is in ˘Normal˘ position.
- ˘H˘: Low Frequency compression

□ Specification

1. Max sound output: 130˘5dB
2. Sound gain: ˘50dB
3. Total harmonic wave distortion: ˘5%
4. Frequency range: 300–4000Hz
5. Input noise: ˘30dB
6. Voltage: d.c. 1.5V
7. Current: ˘4mA

□ Usage

Operation instruction:

Insert the ear plug into ear tightly enough, adjust ˘N-H˘ to H setting, and switch to ON , then adjust the volume knob to degree which you can hear clearly. If you feel that it sounds noisy or hollow, please adjust the N-H to H to compress the low frequency sound.

Important Notes:

If you hear Soaring, please check ear plug if the silicone plug size is applicable, or if it is plug tightly. Make sure to choose applicable size plug, air leakage isn't allowed.

Avoid vibration, falling, humid, high temperature.

Maintenance :

Use dry cloth or cloth with special detergent of hearing aid on, clean the body carefully.

When don't use it, switch the "ON-OFF" knob to "OFF".

□ Maintenance

If you can't hear from the hearing aid, please check following steps:

1. If the power is switched on.
2. If the battery has been used out.
3. If the ear plug is blocked by foreign object.
4. If the earphone is damaged.
5. If the lead is broken.

Keeping Method:

The hearing aid should be kept in the room where is dry, well ventilated and Relative Humidity is less than 80%, no corrosive gas.

□ Rechargeable Instruction diagram



□ Important notes:

1. Lithium battery of the hearing aid, its capacity is 110mAh, it can work for about 25 hours continuously after fully charged.
2. Don't keep the hearing aid in the environment of over high or over low temperature.
3. The battery has its life period, if you find that it doesn't perform well, or can't maintain its usual operation. Change the new battery of same code as replacement.

□ Operation instruction:

1. Connect the larger data cord terminal to Data cord socket of charger, and connect the smaller terminal to charger input socket.
2. Insert the charger plug into power supplier socket.
3. During charging, Red light is on, Green light is also on at the same time. When it is fully charged, the green light will be off.
4. First charging will take 8 hours to get full charging, for future charging, it only takes 2 hours.
5. Must use appointed charger to charge the cell, make sure that the power supply is same as the rated power of charger. The charging time should be less than 12 hours.

□ Important notes of Hearing aid with Li battery:

1. Don't throw the waste hearing aid in fire. Don't use the hearing aid in high temperature.
2. Dismantle and change different battery.
3. If any strange smell or over heat is found, stop using it.
4. Don't use water or organic detergent to clean the hearing aid. Not allowed to put it in water.
5. If don't use it for a long time, fully charge the battery before maintenance.

K-36



A-3

