

Intended Use

Thank you for the purchase of our hearing amplifier. If you it's difficult to hearing very well from others, a hearing amplifier can help you regain your hearing so you can live the life you're used to.

This Bluetooth Hearing Amplifier can be connected directly to a Bluetooth-enabled mobile phone. Whether your phone is an Apple (iOS) or Android system, the hearing amplifier can instantly turn into a Bluetooth headset and turn on the hands-free calling function with a single tap. In addition, through Bluetooth, you can also connect TV, stereo and other audio-visual equipment with Bluetooth function to deliver clear and high-quality audio.

Instruction Diagram



" O " is Power off. " N " is Power on.
 " 0~4 " is Volume grade, " 0 " Min, " 4 " Max

How to use the hearing amplifier

1. Insert earplug into the earphone base. (Figure 1)
2. Change the "O-N" switch to "O" or adjust volume to "1", otherwise the instant big sound will make you uncomfortable.
3. Wear the hearing aid behind your ear, put the earplug tightly in your ear. (Figure 2)
4. Change the "O-N" switch to "N" and adjust volume to get the clear sound.
5. You can turn around the tube to change the earphone, so that you can wear left or right ear. (Figure 3)

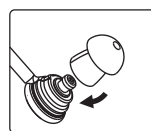


Figure 1



Figure 2

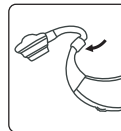


Figure 3

How to use Bluetooth

1. When the hearing amplifier is turned on, press and hold the M mode button for about 3 seconds to hear a beep. And the indicator flashes blue, indicating that the Bluetooth function is enabled and the pairing status is entered. (Figure 4)
2. Start the Bluetooth function of the mobile phone (or other Bluetooth-enabled device), search for the hearing amplifier Bluetooth device, and then select the Bluetooth hearing amplifier model: ZTQ. When the patching is successful, you will hear the "Pairing Successful" voice prompt, and the connection is successful show on the phone. Note: Devices that have been paired successfully will be automatically connected when Bluetooth is turned on for the second time. (Figure 5)

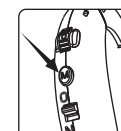


Figure 4

How to use Bluetooth

3. When the mobile phone calls, short press the M mode button to answer/hang up the phone, long press the M mode button for about 3 seconds to reject the connection; when playing music, short press the M mode button to achieve pause/play.
4. When there is no audio signal input to the hearing amplifier, Bluetooth automatically switches to standby state and automatically switches to the hearing amplifier function after 10 seconds. And after a certain period, the Bluetooth function is automatically turned off.


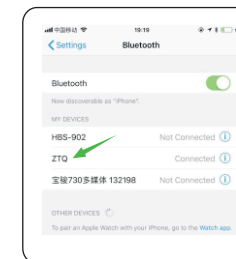


Figure 5

How to charge the hearing amplifier

1. When the sound of the hearing amplifier is very small and the hearing amplifier does not work, the hearing amplifier might have low battery, please charge the hearing amplifier in time.
2. Turn off the hearing amplifier when charge, and insert one end of the DC 5V/50mA~1A charger into the hearing amplifier.
3. Plug the charger into power outlet.
4. The new hearing amplifier should be charged for 6 hours for the first few times you use, for the second time it's better charge after running out of the battery, so necessary after three consecutive cycles, the battery is easy to achieve the best condition.

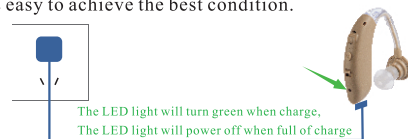

The LED light will turn green when charge.
 The LED light will power off when full of charge

How to charge the hearing amplifier

5. Built in 60mAh lithium battery, it can work continuously for more than 100 hours after it is full charged. (Caution: Please be sure to make our matching battery and charger, don't disassemble and refit battery)

Specification

Item No.	G-25BT
Charger	DC 5V/50mA~1A
Battery	Lithium battery 60mAh
Size	45×40×10mm
Wight	About 6.5g
Max sound output	125 +4dB
Sound gain	38±5dB
Tatal harmonic wave distortion	≤5%
Frequency range	300Hz~4500Hz
Input noise	≤28dB
Voltage	D.C.3.7V
Current	≤2mA

Things to Remember(Cautions)

1. Please don't use the hearing amplifier at a high volume for a long time. It will damage your hearing.
2. Please maintain the product all the time in order to avoid the earplug and microphone opening being blocked by foreign objects.
3. Please don't use when showering, swimming, raining or wet environment, please don't expose the hearing amplifier under sunshine, get close to a stove or other heat sources.
4. Please protect the hearing amplifier carefully and avoid dropping it on the floor.
5. Please use the specified battery and charger exclusively developed for the hearing amplifier.
6. Do not disassemble or modify the hearing amplifier.
7. Keep the hearing aid away from the children.

Things to Remember(Cautions)

8. The plug of adaptor is used as the disconnect device, the disconnect device shall remain readily operable.
9. Attention should be drawn to the environmental aspects of battery disposal.
10. A warning that batteries (battery pack or batteries installed) shall not be exposed to excessive heat such as sunshine, fire or the like.

What to do when you wear the Hearing amplifier device?

1. Adjust volume to the minimum before wearing.
2. Choose an earplug of slightly bigger size to avoid whistler.
3. Increase volume gradually to avoid sudden increase in sound.
4. If you hear a howling, check the ear (silica gel) is appropriate and whether the size of plug is tight, the appropriate choice of earplugs and plugged, make sure no air leakage.
5. Keep the ear plug clean.
6. Remove the battery if you don't use it for long time.

USER GUIDE



