


Function

Thank you very much for buying the sound amplifier.
When you see film, watch the play, have meeting, or have classes, the product can help you to hear the sound clearly and loudly. While in forest, if you want to hear very slight sound, it will be a good choice.

Specification

1. Max sound output: $\leq 128+4\text{dB}$
2. Sound gain: $38 \pm 5\text{dB}$
3. Total harmonic wave distortion: $\leq 5\%$
4. Frequency range: 300~4000Hz
5. Input noise: $\leq 30\text{dB}$
6. Voltage: d.c. 1.5V
7. Current: $\leq 4\text{mA}$

Instruction diagram

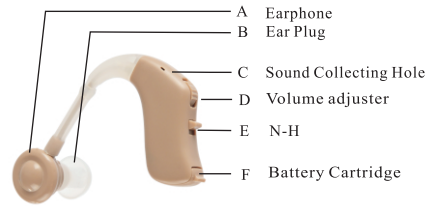

"N": Normal
"H": lower the noise & weaken the sound of low frequency





Power On



Power Off

Usage

Open the cartridge cover, put the AG5 or A13 battery in Correct direction. Zinc Air Battery is best choice and have long life .

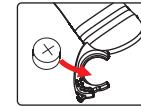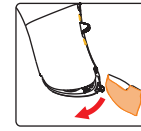


1. Make sure to turn off the power or adjust the earphone to the slightest sound before use.

2. Put the ear hook on your ear, plug in the ear plug tightly.


(Note: The unit must be on the Power off position and turn the earphone down to the lowest sound, otherwise the instant sound will make you uncomfortable).

Usage

3. Turn on the power, and adjust the sound volume to get clear sound.

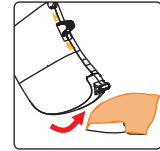

4. Check if the ear plug has been plugged tightly once you hear the whistling sound. please change the plug of other size if it can't be plugged tightly.

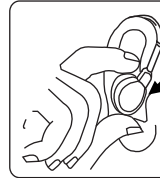

Maintenance

No sound from sound amplifier.

Check as follows:

1. If you turn on the power.
2. If you fit the battery in correct direction.
3. If the battery power has been used up.
4. If the earphone is damaged.
5. If the lead broken.

The sound from sound amplifier is very weak and vague.

1. Battery power isn't enough.
2. If something is wrong with earphone.

Use dry cloth and special detergent to clear unit body.

Note

1. It is forbidden to be used by those patients with tympanitis or ear pyogenesis.
2. Don't use it at high volume for long time. Otherwise it will injure your ear. If you feel uncomfortable, please stop using.
3. Please maintain the products all the time, avoid ear plug and sound collecting hole being blocked by foreign objects. It will affect the result of sound amplifier.
4. Avoid shocking, falling the product. Don't keep or use it in environment of humid and high temperature.

Note

5. Don't throw the scrapped battery as common rubbish. Should be handled by special equipment.
6. Please take out the battery if not used for long time. Otherwise the components will be rotted by the rotten liquid from battery.
7. Don't use the product in potential explosive or dangerous environment.
8. Keep the unit away from children. It will injure him or others, or damage the unit.

Operation Manual

